

2-in-1 WEDGES & ACID REFLUX WEDGES

Item #: CBW, 30-960R, 30-970R, 30-980R

Go to www.contourliving.com/get-product-help/ to see inflation/deflation instructions.

3 Easy Steps to Elevated Comfort


1 Inflate Your Wedge with the Hand Pump

Open valve cap and place pump into opening. Push / pull pump to fully inflate.



2 Adjust Air Pressure to Suit Your Comfort

Lay on wedge and **press on the flat tab of the Air Release Switch** until wedge reaches desired comfort level.

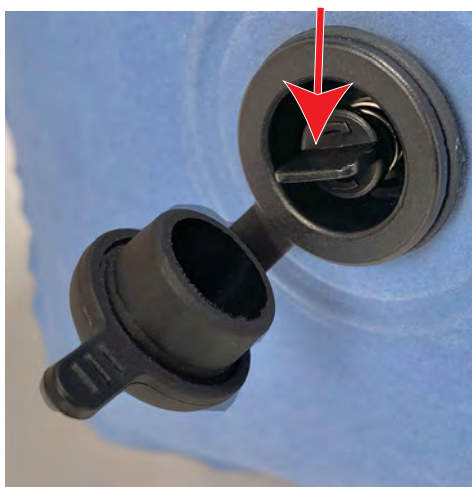

3 Close the Cap

Close the valve cap tightly to prevent air leaks.



To Deflate

Gently twist the **Air Release Switch** a 1/4-turn until it clicks into open valve position. Let all air out. Tightly close the valve cap.



Enjoy 2 Positions





2-IN-1 WEDGES & ACID REFLUX WEDGES

Item #: CBW, 30-960R, 30-970R, 30-980R

To Clean

Spot clean only with soap and water. Do not use chemicals.



Important Safety Information

Read and follow all warnings and instructions before using this product. Injury can result from improper use. Save these instructions.

WARNING

- **DO NOT USE AS A FLOTATION DEVICE. NOT INTENDED FOR USE IN WATER.**
- **THIS IS NOT A TOY. DO NOT ALLOW CHILDREN TO STAND OR JUMP ON BACK WEDGE.**
- **DO NOT USE AS A BED FOR INFANTS, CHILDREN OR PETS.**
- **USE ONLY FOR THE INTENDED PURPOSE - TO ELEVATE TORSO OR LEGS.**

CAUTION

- **INFLATE WITH HAND PUMP ONLY. DO NOT USE A COMPRESSOR WHICH COULD CAUSE THE WEDGE TO BURST.**
- **IT IS NORMAL FOR INFLATABLE ITEMS TO LOSE AIR OVER TIME. CHECK YOUR WEDGE PERIODICALLY TO SEE IF IT NEEDS FURTHER INFLATION.**
- **THE MATERIAL OF THIS ITEM IS SENSITIVE TO COLD AND MAY CRACK IF LEFT IN 40°F OR BELOW TEMPERATURES.**
- **KEEP SHARP OBJECTS AWAY FROM THE 2-IN-1 COMFORT WEDGE TO PREVENT PUNCTURES.**
- **DO NOT PLACE ON ROUGH SURFACES SUCH AS CONCRETE.**